




# iCassette™ Drug Test

## Drug Test Interpretation Quiz

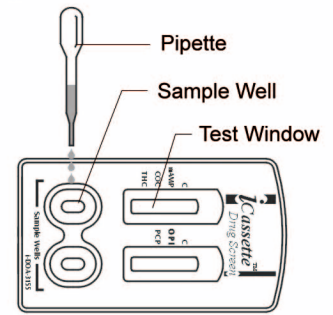

### I. MULTIPLE CHOICE

- When collecting specimen in a cup, what should the temperature range be?
  - 90-100° Celsius
  - 85-95° Fahrenheit
  - 90-100° Fahrenheit
  - 85-95° Celsius

ANSWER: \_\_\_\_\_

- The absence of a line next to a drug abbreviation indicates a \_\_\_\_\_ for that particular drug.
  - Non negative
  - Negative
  - Invalid
  - All of the above

ANSWER: \_\_\_\_\_

- When collecting specimen, transfer \_\_\_\_\_ full drops of urine to each specimen well of the test device.
  - 3
  - 5
  - 2
  - None of the above

ANSWER: \_\_\_\_\_

- Negative results may be interpreted as soon as all lines are visible. Non negative results should only be interpreted at \_\_\_\_\_.
  - 10 minutes
  - 1 minute
  - 3 minutes
  - 5 minutes

ANSWER: \_\_\_\_\_

### II. MATCHING (Match the appropriate abbreviation to the drug name)

- |                     |       |      |
|---------------------|-------|------|
| 1. Marijuana        | _____ | COC  |
| 2. Amphetamines     | _____ | PCP  |
| 3. Cocaine          | _____ | mAMP |
| 4. Opiates/Morphine | _____ | AMP  |
| 5. Phencyclidine    | _____ | OPI  |
| 6. Methamphetamine  | _____ | THC  |

Please fill out your contact information to receive a Certificate of Completion:



Name \_\_\_\_\_

Company \_\_\_\_\_

Address \_\_\_\_\_

Phone (\_\_\_\_\_) \_\_\_\_\_

### III. IDENTIFY THE RESULTS AS EITHER: NON NEGATIVE, NEGATIVE, OR INVALID

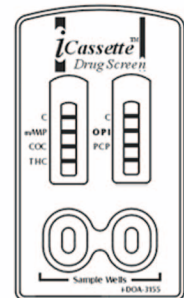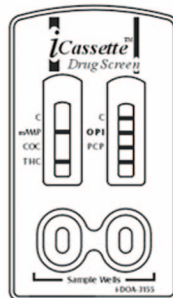

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_