
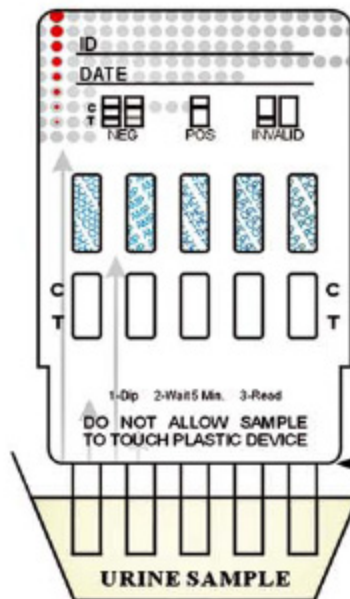


# Instructions

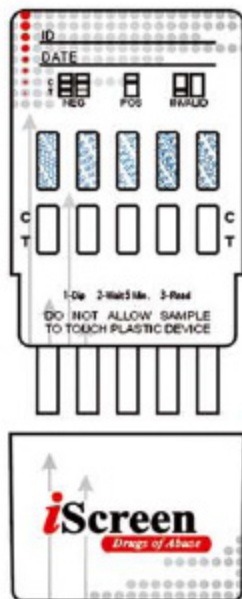

This illustration shows a 5-Drug Test Card

Do not allow specimen to touch the device housing

1. Remove iScreen<sup>®</sup> test from foil pouch at room temperature. Take off the cap revealing the testing strips.
2. Immerse test vertically in the urine sample for at least 10 - 15 seconds. Do not allow the specimen to touch the device housing.
3. Read at five (5) minutes.  
Results stable up to four (4) hours.



## Interpretation of Results:



**Negative**

(2 Lines - Any indication of a line in both test and control fields is to be interpreted as a **NEGATIVE**)



**Non-Negative**

(1 Line)



**Invalid**

(No line(s) or no control line(s))