


## Instant Bath Salts Drug Test Procedure (Instructions)

### INSTRUCTIONS FOR USE

- 1) Remove the dip card from the foil pouch.
- 2) Remove the cap from the dip card. Label the device with patient or control identifications.
- 3) Immerse the absorbent tip into the urine sample for 5 seconds. Urine sample should not touch the plastic device.
- 4) Replace the cap over the absorbent tip and lay the dip card on a clean, flat, and non-absorptive surface.
- 5) Read result at 5 minutes. **DO NOT INTERPRET RESULT AFTER 10 MINUTES.**

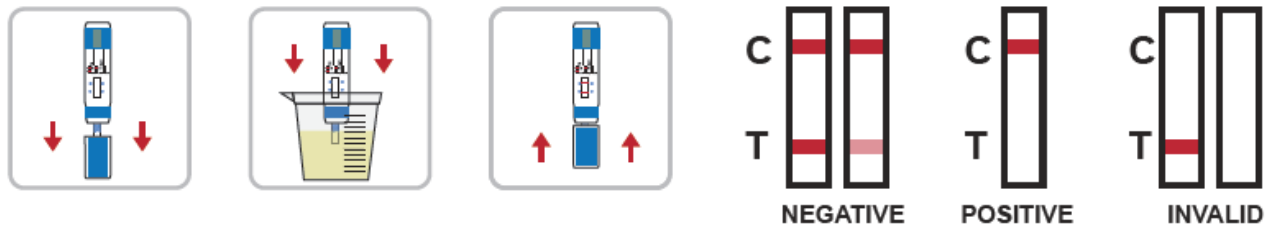

### INTERPRETATION OF RESULTS

(Please refer to the previous illustration)

**NEGATIVE:** \*Two colored lines appear, one in the control region (C), and another one in the adjacent test region (T). This negative result indicates that the drug concentration is below the detectable level.

\*NOTE: The shade of color of the line(s) may vary, but it should be considered negative whenever there is even a faint distinguishable color line.

**POSITIVE:** One colored line appears in the control region (C). No line appears in the test region (T). This positive result indicates that the drug concentration is above the detectable level.

**INVALID:** Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test using a new test device. If the problem persists, discontinue using the lot immediately and contact your supplier.