


## Instant K2 Drug Test Procedure (Instructions)

### INSTRUCTIONS FOR USE

- 1) Remove the test device from the foil pouch.
- 2) Remove the cap from the test device. Label the device with patient or control identifications.
- 3) Immerse the absorbent tip into the urine sample for 5 seconds. Urine sample should not touch the plastic device.
- 4) Replace the cap over the absorbent tip and lay the device flatly on a non-absorptive clean surface.
- 5) Read result at 5 minutes

**DO NOT INTERPRET RESULT AFTER 10 MINUTES.**

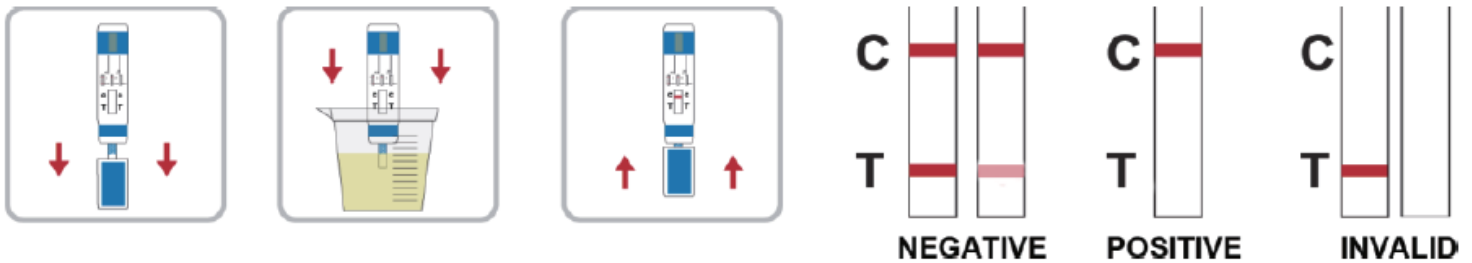

### INTERPRETATION OF RESULTS

(Please refer to the previous illustration)

**NEGATIVE:** Two lines appear. \* One color line should be in the control region (C), and another apparent color line adjacent should be in the test region (T). This negative result indicates that the drug concentration is below the detectable level.

\*NOTE: The shade of color in the test line region (T) will vary, but it should be considered negative even if there is a faint distinguishable color line.

**POSITIVE:** One color line appears in the control region (C) while no line appears in the test region (T). This positive result indicates that the drug concentration is above the detectable level.

**INVALID:** Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test using a new test device. If the problem persists, discontinue using the lot immediately and contact your supplier.