


Procedure Card for Drugs of Abuse Cup

(Cups available with adulteration)

For detailed instructions please read the package insert. Store at 2-30°C (36-86°F) in the original sealed pouch. Do not freeze.

CUP COMPONENTS

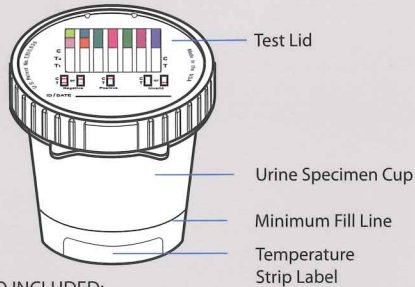

ALSO INCLUDED:


- Package insert
- Security seal (if applicable)
- Adulteration color chart (if applicable)

COLLECTION & TESTING PROCEDURE

- 1 Collect the specimen. Sample must meet minimum urine level as indicated on the cup.




- 2 Secure test device cap to the specimen cup. NOTE: Screw the cap tightly to ensure adequate seal.

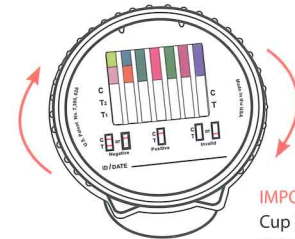


IMPORTANT:
Cup lid must be secured tightly by twisting lid a quarter turn **AFTER** lid is snug

- 3 Turn cup on its side to activate testing. Place on a flat surface.

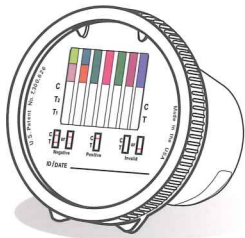

- 4 When applicable, read adulteration results by visually comparing color of reagent pads to corresponding blocks on the Color Chart at the time indicated.

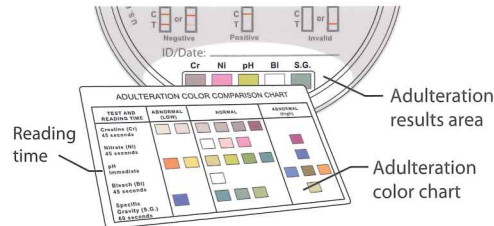

- 5 Read results at 5 minutes. Do not interpret result after 10 minutes.

